INSTRUCTIONS FOR USE


The Securject Pro is an easy-to-use Professional lancing device for obtaining blood from the fingertip. There are five different penetration depths to choose from to obtain the optimum blood volume for the skin type of the patient and test type required.

The Securject Pro has been specially developed for use by healthcare professionals for blood sample collection from multiple patients. The Securject Pro has been designed to include all of the safety, ease-of-use and hygiene requirements that multiple patient environments and healthcare professionals may have.


The Securject Pro includes an integral platform with Comfort Zone Technology™ which is ejected with the lancet in a single motion after completing the lancing, ensuring those parts of the system that come into contact with the patient's skin or blood are always replaced. Exclusively for use with the SecurLancets Pro, the Securject system ensures a virtually painless sample is obtained safely.

Please read the following instructions carefully before using and retain for future reference. Failure to follow the instructions correctly may lead to needle stick injury.

Contents: Securject Pro - Instructions for use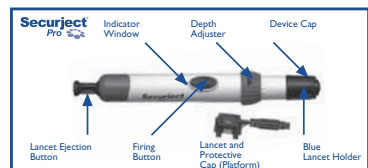


Select the Penetration Depth
There are 5 penetration depth settings on the Securject Pro indicated by drops on the body. The smallest drop represents the shallowest depth and the largest drop represents the greatest depth. Adjust the penetration depth of the device by turning the green depth adjuster until the blue arrow lines up with your selected depth; you will feel a click as each depth is set. (Fig. 1)

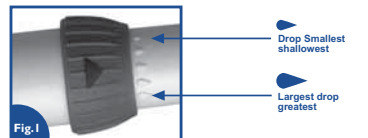
Insert a new SecurLancet Pro
Remove the device cap and place a new SecurLancet Pro between the arms of the lancet holder. Push down firmly as far as it will go; once the lancet is fully loaded, you will hear a click and the blue lancet ejection button will be visible. (Fig. 2)

Attach the Platform
Remove the platform from the lancet by twisting gently. (Fig. 3)
Holding the Securject Pro in one hand and the SecurLancet Pro platform in the other, push the platform onto the front of the device to cover the lancet (Fig. 4).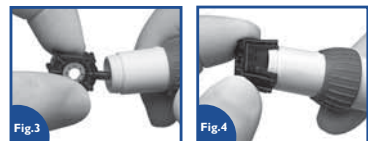

Safety Check
Check there is nothing visible in the indicator window. If a red dot is visible, eject the lancet and insert a new SecurLancet Pro as before. (Fig 5A)
The Securject Pro must not be used if there is a red dot visible in the window until a new lancet has been inserted.

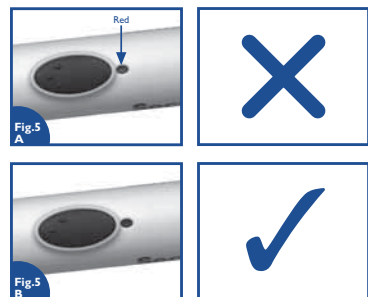Obtain the Sample
Hold the Securject Pro against the side of the patient's finger and press gently.
Press the blue firing button on the main body of the device to complete the lancing and then remove the Securject Pro from the finger. (Fig. 6)
Wait a moment to allow blood to flow freely to the puncture site. If the blood drop is too small, gently massage the tip of the finger until a sufficient sample is collected.

Eject and Dispose of the Lancet
Hold the Securject Pro pointing downwards over a suitable sharps container. Press the blue lancet ejection button at the back of the device and the used lancet and platform will automatically be ejected from the device. (Fig. 7)
This ensures that all parts of the device that have come into contact with the skin or blood of the patient are always replaced.
Once completed, replace the device cap and the Securject Pro is ready for use again with a new SecurLancet Pro.

Comfort Zone Technology™
Comfort Zone Technology™ increases comfort when lancing by targeting the sensation and perception of pain. This unique approach disguises the lancing action to provide a more comfortable sample.

Cleaning and Maintenance
When required, use a soft cloth with mild soap and water to wipe the outside of the Securject Pro and allow to dry before storing. Do not immerse the Securject Pro in liquid. Do not use any other method of cleaning the device.

Safety Information
The SecurLancet Pro is for single-use only. Do not use if the platform has become detached. All SecurLancet Pro should be disposed of in a suitable sharps container.
The Securject Pro is a re-usable device for use with more than one patient. The Securject Pro should only be used with SecurLancet Pro.

Warranty
The Securject Pro is guaranteed against manufacturing defects for two years from date of purchase. In the unlikely event of failure, please contact A. Menarini Diagnostics Ltd. - Wharfedale road, Wokingham, Wokingham, Berkshire RG41 5RA - Tel.: +44 1189 441100 - Fax: +44 1189 444111. This does not affect your statutory rights.

Manufactured by Owen Mumford Ltd, Brook Hill, Woodstock, OX20 1TU, England for A. Menarini Diagnostics, Firenze, Italy.

Securject Pro is a registered trademark of A. Menarini Diagnostics

Comfort Zone Technology™ is a registered trademark of Owen Mumford Ltd.

SECURITY INFORMATION
This product conforms to European Directive 93/42/EEC

SYMBOLS	
	Temperature limitation
	Manufacturer
	Catalogue number
	Read the instructions carefully
	This product conforms to European Directive 93/42/EEC
	Batch code
	Use by

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Produsert av Owen Mumford Ltd., Brook Hill, Woodstock, OX20 1TU, England for A. Menarini Diagnostics, Firenze, Italia.
Velbehagsområde – teknikk er et registrert varemerke hos Owen Mumford Ltd.